CRADLE

For use with Classic Stepper

REFERENCE GUIDE

Best[®] Sonalis[™] TRT TriView

BK Medical 8558, 8658, 8848, E14CL4b

Esaote TRT33

GE Healthcare ERB, E7C8L

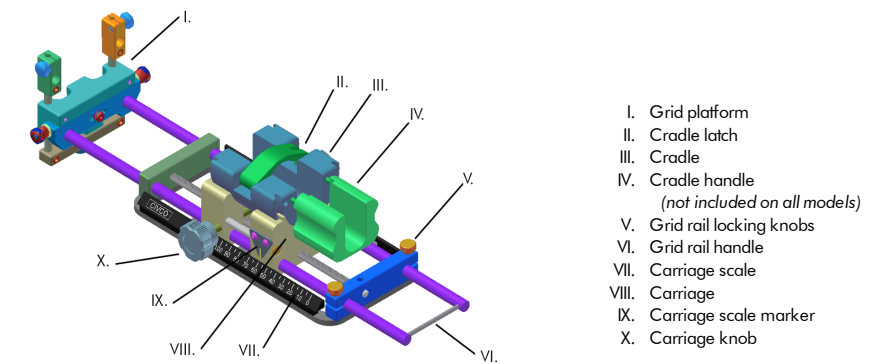
Hitachi Aloka Medical C41L47RP, EUP-U533, UST-672-5/7.5

Siemens Endo-Pil


CE


Symbol	Title of Symbol	Description of Symbol
	Manufacturer (ISO 15223-1, 5.1.1)	Indicates the medical device manufacturer.
	Authorized Representative in the European Community (ISO 15223-1, 5.1.2)	Indicates the Authorized Representative in the European Community.
	Date of manufacture (ISO 15223-1, 5.1.3)	Indicates the date when the medical device was manufactured.
	Batch code (ISO 15223-1, 5.1.5)	Indicates the manufacturer's batch code so that the batch or lot can be identified.
	Catalog number (ISO 15223-1, 5.1.6)	Indicates the manufacturer's catalogue number so that the medical device can be identified.
	Serial number (ISO 15223-1, 5.1.7)	Indicates the manufacturer's serial number so that a specific medical device can be identified.
	Consult instructions for use (ISO 15223-1, 5.4.3)	Indicates the need for the user to consult the instructions for use.
	European Conformity (EU MDR 2017/745, Article 20)	Indicates manufacturer declaration that the product complies with the essential requirements of the relevant European health, safety and environment protection legislation.
	Medical Device (MedTech Europe Guidance: Use of Symbols to Indicate Compliance with the MDR)	Indicates the product is a medical device.
	Quantity (IEC 60878, 2794)	To indicate the number of pieces in the package.

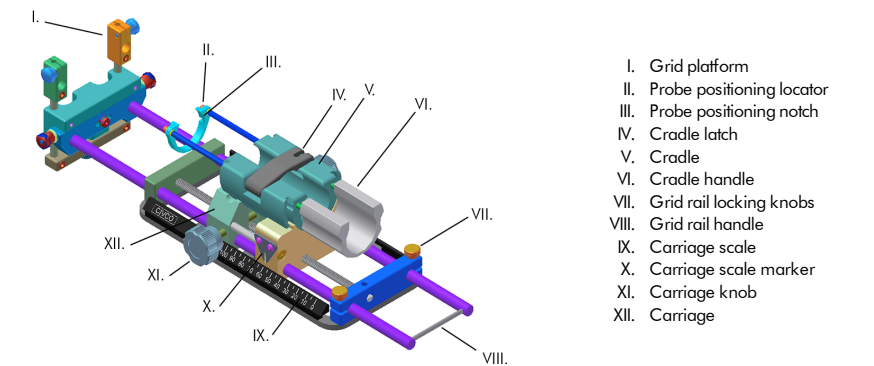Esate TRT33
GE Healthcare ERB, E7C8L
Hitachi Aloka Medical C41L47RP, EUP-U533, UST-672-5/7.5
Siemens Endo-PII

BK Medical 8558, 8658, 8848, E14CL4b

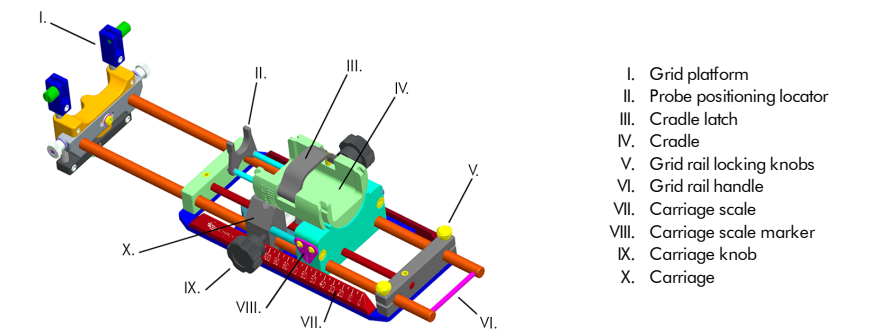INTENDED USE

The equipment is intended to hold and manipulate ultrasound imaging probes, and report position, during prostate brachytherapy, cryotherapy, transperineal template-guided biopsy, and/or fiducial marker placement procedures (including volume determination of the prostate gland), and/or the application of radionuclide source(s) into the body during brachytherapy.

INDICATIONS FOR USE

- Prostate - Diagnostic imaging and minimally invasive puncture procedures.
- Surgical (Prostate) - Diagnostic imaging and puncture procedures.

PATIENT POPULATION

Equipment is for use in adult males with suspected or diagnosed prostate cancer.

INTENDED USERS

Equipment should be used by clinicians medically trained in ultrasound imaging. User groups may include, but are not limited to: Physicists, Radiation Oncologists, Surgeons and Urologists.

PERFORMANCE CHARACTERISTICS

- Stepper allows for incremental longitudinal and rotational movement of the ultrasound transducer to achieve its intended purpose.
- Stepper includes a quick mounting, secure connection to CIVCO stabilizers to ensure stability of the ultrasound transducer.

NOTE: For a summary of clinical benefits for this product, visit www.CIVCO.com.

CAUTION

Federal (United States) law restricts this device to sale by or on the order of a physician.

WARNING

- *Before use, you should be trained in ultrasonography. For instructions on the use of your transducer, see your system's user guide.*
- *Prior to use inspect device for signs of damage, if damage is evident do not use.*
- *Cradle assembly is packaged non-sterile and is reusable. To avoid possible patient contamination, ensure equipment is properly cleaned and disinfected before each use. See reprocessing section for instructions on how to properly clean and disinfect.*
- *If the product malfunctions during use or is no longer able to achieve its intended use, stop using the product and contact CIVCO.*
- *Report serious incidents related to the product to CIVCO and the competent authority in your Member State, or appropriate regulatory authorities.*

NOTE: Product is not made with natural rubber latex.

ATTACHING CRADLE

1. Open the cradle latch.
2. **For Esaote TRT33, GE Healthcare ERB, E7C8L transducers:**
Place and orient transducer into the cradle, sliding transducer forward until it stops.
For Hitachi Aloka Medical UST-672-5/7.5, Best® Sonalis™ TRT TrnView transducers:
Place and orient transducer into the cradle.
For Hitachi Aloka Medical C41L47RP, EUP-U533, Siemens Endo-Pil transducers:
Slide transducer back into the cradle until it stops.
For BK Medical 8558, 8658, 8848, E14CL4b transducers:
Slide cable end of transducer back into the cradle, ensuring locating feature fits into notch.
3. Close cradle latch and tighten knob.
4. Replace cradle with transducer back into stepper by rotating into carriage. Center transducer in rotation by feeling the center line detent during rotation.
5. Position carriage in a central location on carriage scale by turning carriage knob until carriage scale marker is at 60.

6. Loosen grid rail locking knobs and pull back on grid rail handle so grid platform is out of the way for initial transducer positioning.

FUNCTIONAL PREUSE CHECKS

WARNING

- Prior to each use perform the following checks to ensure optimal stepper performance.

1. Rotate cradle to ensure movement through full range of rotational travel.
2. If cradle does not move freely, apply lubricant that complies with hospital policies and procedures. Super Lube® Multi-Purpose Synthetic Grease with Syncolon® (PTFE) is recommended.

REPROCESSING

WARNING

- Users of this product have an obligation and responsibility to provide the highest degree of infection control to patients, co-workers and themselves. To avoid cross-contamination, follow infection control policies established by your facility.
- See your system's user guide for reprocessing transducer between uses.
- These procedures for reprocessing have been validated for effectiveness and compatibility. Equipment could be damaged or cross-contaminated due to improper reprocessing.
- Do not place in a mechanical washer.
- Do not use ethylene oxide or autoclave to sterilize.

CLEANING

1. Prepare an enzymatic detergent solution, such as Enzol® Enzymatic Detergent, at manufacturer's recommendation.
2. Fully immerse Classic Stepper in prepared enzymatic detergent solution. Soak for 3 minutes.
3. After soak time, brush entire surface for 1 minute. Ensure all crevices and grooves are brushed.
4. Dispose of used enzymatic detergent solution and prepare a new enzymatic detergent solution, such as Enzol® Enzymatic Detergent, at manufacturer's recommendation.
5. Allow Classic Stepper to soak in new enzymatic detergent for 3 minutes.
6. Remove Classic Stepper from enzymatic detergent solution and rinse under running tap water for a maximum of 1 minute, but not less than 50 seconds.
7. Dry Classic Stepper with a soft, clean, lint-free cloth.

- NOTE:**
- If there is residual water inside the stepper, compressed air may be used at a maximum of 150 psi to remove remaining water.
 - Prolonged exposure of anodized aluminum and nylon to Enzol® Enzymatic Detergent may cause slight discoloration of surfaces.

DISINFECTION

1. Unfold a clean germicidal wipe with at least 55% isopropyl alcohol and 0.5% quaternary ammonium, such as Super Sani-Cloth® Germicidal wipe, and thoroughly wet surface.
2. Allow treated surface to remain wet for no more than 2 minutes. Use additional wipe or wipes, if necessary, to ensure continuous 2 minutes wet contact time.
3. Once 2 minute wet contact time has been reached, allow surfaces to air dry.

STORAGE AND DISPOSAL


WARNING

- Dispose of single-use components as infectious waste. Reprocess reusable components after each use according to instructions for use.


- NOTE:**
- Device should be stored to keep all components together and secure.
 - For questions or to order additional CIVCO products, please call +1 319-248-6757 or 1-800-445-6741 or visit www.CIVCO.com.
 - All product to be returned must be in its original packaging. Contact CIVCO for further instructions as needed.

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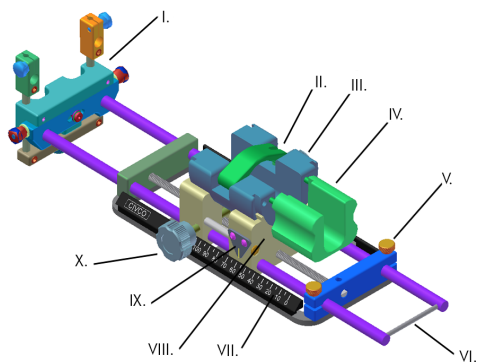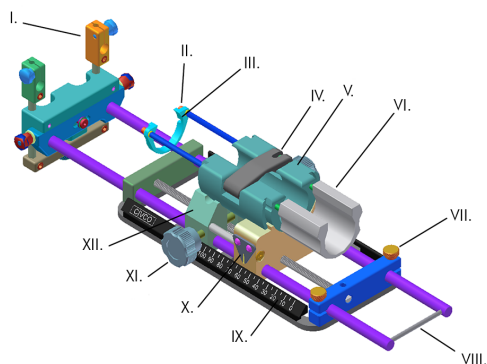
Esaote TRT33
 GE Healthcare ERB, E7C8L
 Hitachi Aloka Medical C41L47RP, EUP-U533, UST-672-5/7.5
 Siemens Endo-PII

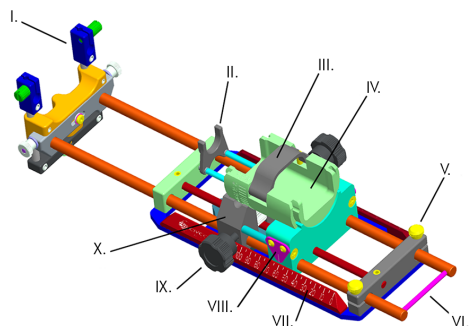- I. Rešetkasta platforma
- II. Lokator za pozicioniranje sonde
- III. Urez za pozicioniranje sonde
- IV. Zasun podloge
- V. Podloga
- VI. Ručka podloge
- VII. Gumbi za zaključavanje tračnice rešetke
- VIII. Ručka tračnice rešetke
- IX. Skala za kolica
- X. Oznaka skale za kolica
- XI. Gumb za kolica
- XII. Kolica

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COPYRIGHT © 2020 ALL RIGHTS RESERVED. CIVCO IS A REGISTERED TRADEMARK OF CIVCO MEDICAL SOLUTIONS. ALL OTHER TRADEMARKS ARE PROPERTY OF THEIR RESPECTIVE OWNERS. PRINTED IN USA. ALL PRODUCTS MAY NOT BE LICENSED IN ACCORDANCE WITH CANADIAN LAW.
MPS GmbH
CIVCO Medical Instruments Co., Inc.